

GENESIS




HSD

Magnetotherapy is a form of physiotherapy that uses electromagnetic energy, stimulating tissue regeneration. It also acts on bone tissues, lymphatic tissues and muscles, accelerating trophism and promoting the rehabilitation process.


Ultrasound therapy, with the stimulation current, is one of the most common forms of physical therapy treatment. Ultrasound therapy is classified as mechanical thermal therapy. According to the therapy parameters (frequency, type of emission, intensity, duration and method of therapy) we can obtain a thermal effect or a micro-massage in the segments of the treated tissue. The thermal increase affects not only the superficial but also the deep structures

Foot Reflexology Chart

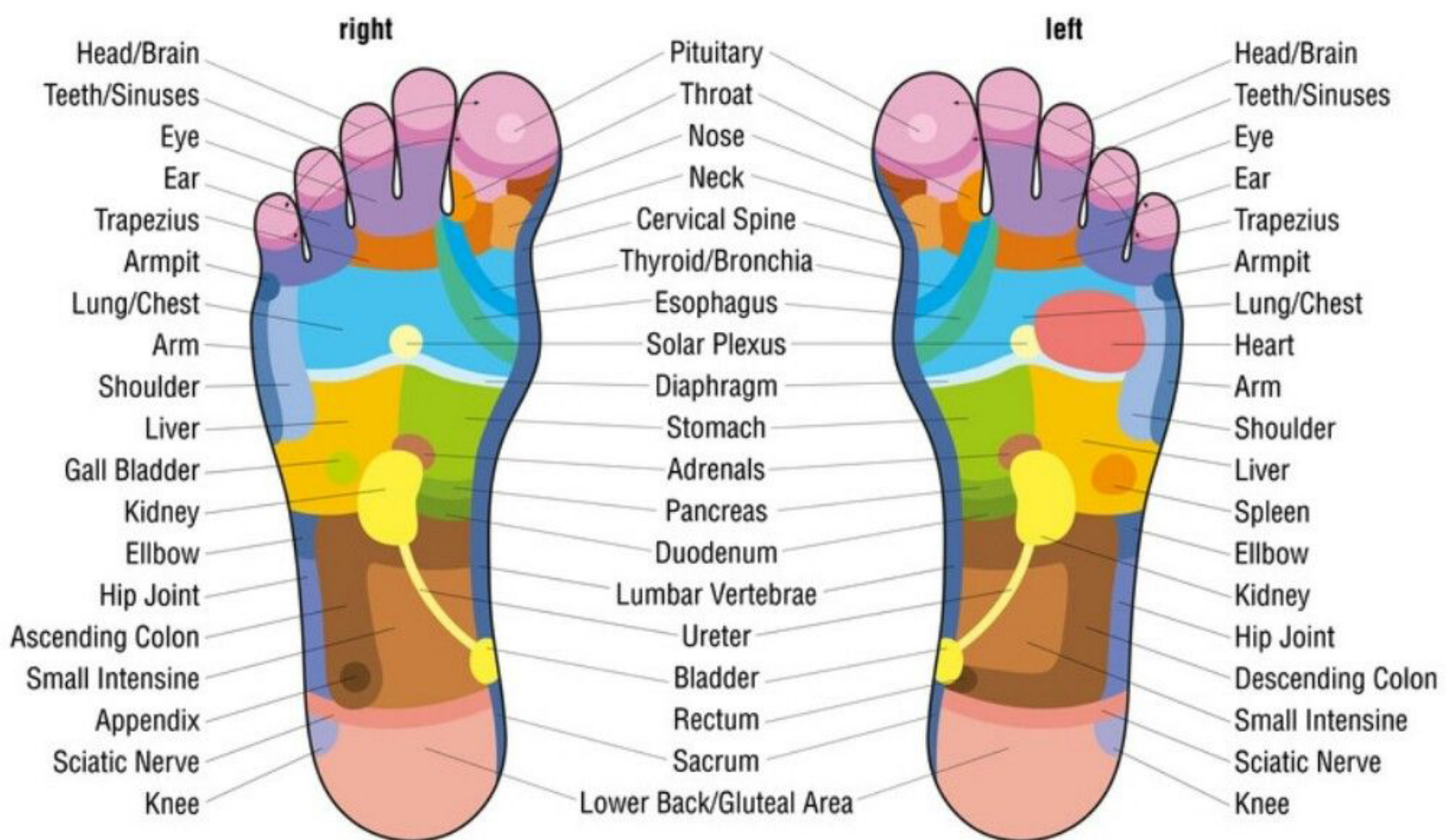

GENESIS PROGRAMS/PATHOLOGY

- 01 HARD TOE - 02 HALLUX VALGUS - 03 RETROACHILLEA BUNS - 04 BLOOD CIRCULATION 05
 HAMMER FINGERS - 06 PLANTAR FASCIITIS - 07 METATARSALGIA - 08 MORTON NEUROMA
 09 PATHOLOGIES OF THE LATERAL COMPARTMENT OF THE ANKLE - 10 HOLLOW FOOT
 11 DIABETIC FOOT - 12 CHARCOT FOOT - 13 PARALYTIC FOOT - 14 FLATFOOT
 15 RHEUMATIC FOOT - 16 HAGLUND'S SYNDROME - 17 TARSAL TUNNEL SYNDROME
 18 HEEL SPUR - 19 HEEL PAIN - 20 TENDINITIS OF THE EXTENSORS
 21 TENDINITIS OF PERONIERI - 22 YACHELE TENDINITIS - 23 PROXIMAL TENDINITIS OF THE
 ANTERIOR TIBIAL - 24 ULTRASOUND - 25 MAGNETOTHERAPY

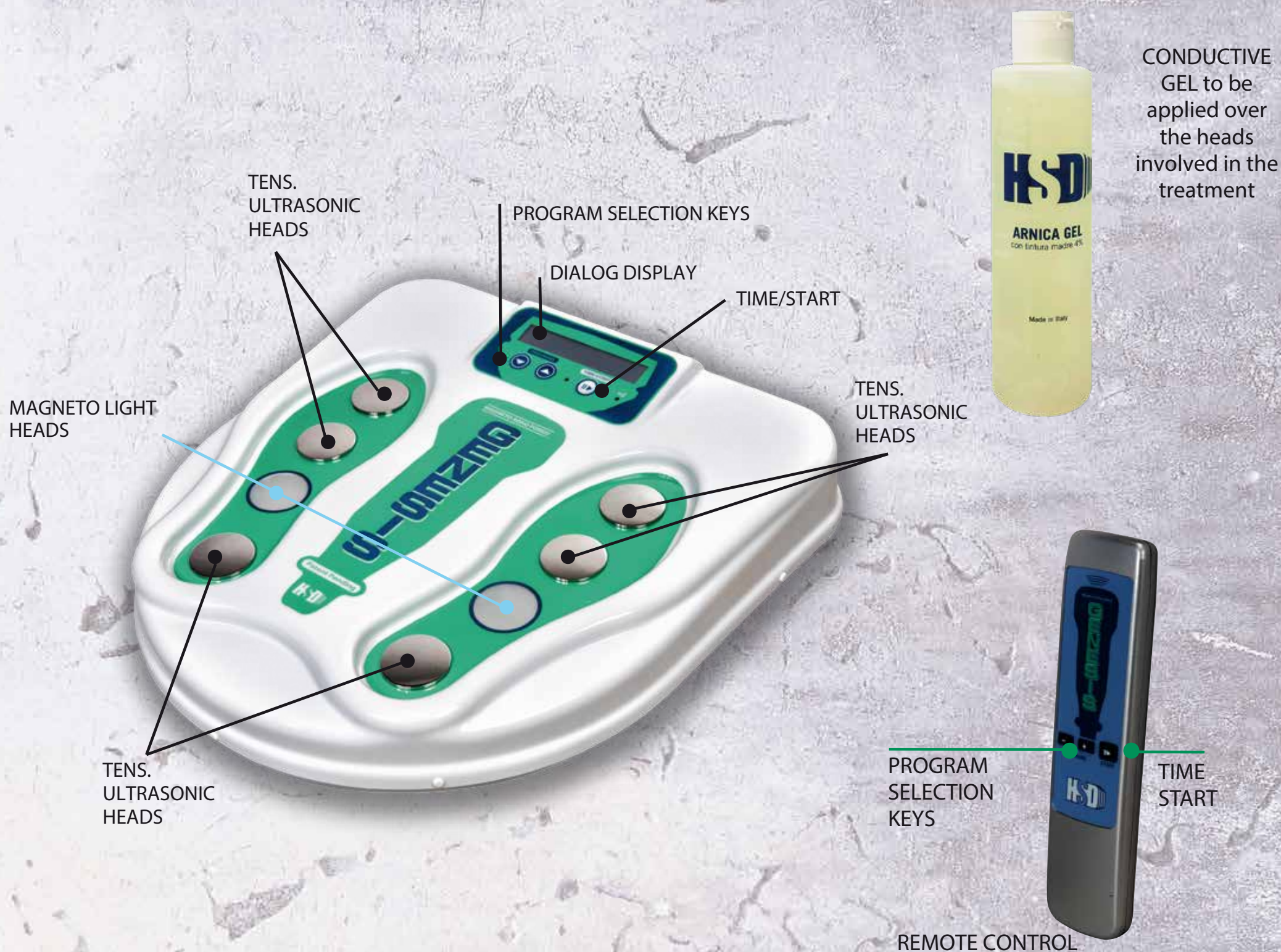

CONDUCTIVE GEL to be applied over the heads involved in the treatment

GENESIS USE OF THE APPLIANCE

Remove the device and place it on a stable horizontal rigid surface, and near a power outlet. Check that the voltage of the power supply corresponds to the voltage and frequency on the appliance plate. Before inserting the power cable into the socket, make sure that the power switch is set to 0 (zero). When turned on, bringing the switch to position 1 turns on the DISPLAY on which the software presentation message appears.

At the end of the previous display, SELECT A PROGRAM appears (if you do not choose a program, the last run is proposed again)

At this point acting on the keys ▲ ▼ you can select one of the available programs. With the same keys, the TENS is adjusted when the program has started. After having chosen the program, press the TIME / START key to start the treatment, to pause it, press the same button, while keeping the TIME / START button pressed, the treatment is definitively terminated. The BEEP emits 3 notes close together, the STOP LED lights up and the DISPLAY shows the message PROGRAM ENDED. Afterwards you can choose another program.

TECHNICAL DATA.

Power supply: 230VAC 50Hz - Power: 50W - Working time: 20 min.

On the back and on the ultrasound head are the identification plates with the technical data. CE marking.



HSD

STRADA ACQUASALATA N. 11-INT. 6 - 47899 - SERRAVALLE (RSM)
www.hsd.sm / info@hsd.sm

Tel. 0549.941496

 **NUMERO VERDE**
8000 200 55